






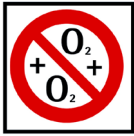



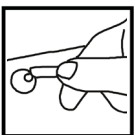






















Oxygen concentrator Instruction card

| | | |
|---|--|---|
| <p>1 Always read the instructions.</p>  | <p>2 No open fire. Do not smoke. No e-smoker or vapes either.</p>   | <p>3 Do not use oil or greased based products.</p>  |
| <p>4 Ventilate the room well.</p>  | <p>5 For fire safety reasons, do not use electrical devices that can generate sparks in the room where the oxygen concentrator is placed.</p>  | <p>6 Place the concentrator in a safe location as indicated by the service staff. The concentrator must always be used in an upright position.</p>  |
| <p>7 Do not place the concentrator under a coat rack or against curtains. This is to prevent oxygen saturation.</p>  | <p>8 Do not use any extension cords.</p>  | <p>9 Technical maintenance must be performed by our service staff only.</p>  |
| <p>10 Plug the power cord into the outlet and turn on the concentrator using the on-off switch.</p>  | <p>11 Slide the long oxygen tube over the gray nozzle. Check if the tube is securely attached.</p>  | <p>12 Set the flow meter as prescribed by your physician.</p>  |
| <p>13 Check the flow by placing the nasal cannula and/or tube in a glass of water. In the absence of bubbles, inspect the hose and flow meter. In the absence of a flow, contact Westfalen Medical.</p>  | <p>14 Turn off the concentrator after use. Do not run the concentrator when the device is not in use.</p>  | <p>15 Maximum length of connection tube.</p>  |
| <p>16 We recommend replacing the nasal cannula once a week.</p>  | <p>17 The coarse filter should be cleaned weekly. It can be found on the side and/or back of the concentrator.</p>  | <p>18 Malfunctions and/or questions: Tel. 0570-858 450</p>  <p>Westfalen Medical</p>  |

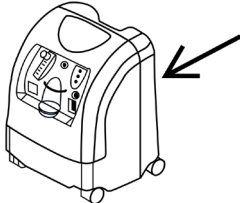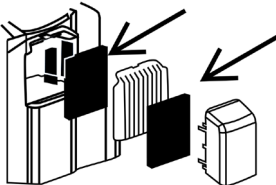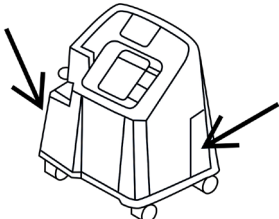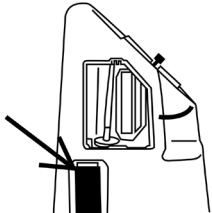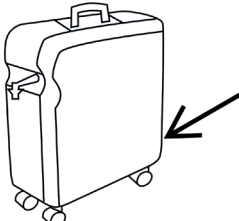

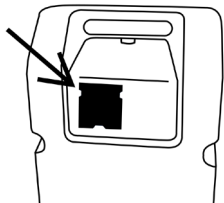Filters Instruction card

| | |
|--|---|
| <p>1 Filters are located at the back.</p> <p>Perfect O2</p>  | <p>2 Vacuum the coarse filter weekly.</p>  |
| <p>1 Filters are located on both sides.</p> <p>Platinum S/9</p>  | <p>2 Vacuum coarse filters on both sides weekly.</p>  |
| <p>1 Filters are located on both sides.</p> <p>Kröber O2 / 4.0</p>  | <p>2 Vacuum the coarse filter weekly.</p>  |
| <p>1 Filters are located at the back.</p> <p>DeVilbiss 1025 / C525</p>  | <p>2 Vacuum the coarse filter weekly.</p>  |
| <p>1 Filters are located at the back.</p> <p>Everflo</p>  | <p>2 Vacuum the air intake weekly.</p>  |

For instructional videos, visit our website: zuurstofinfo.nl/downloads/instructievideos-zuurstofconcentratoren

